



OWNER'S MANUAL

MEDEVAC RESCUE LITTER P/N 402-TIS

WARNING: THIS LITTER SHOULD ONLY BE USED BY PERSONS WITH SUFFICIENT OPERATIONAL AND MEDICAL TRAINING TO ENSURE PROPER PATIENT HANDLING.




1. Lay the Medevac case horizontally with instruction pocket facing up. Unzip the case and remove the folded litter. Place litter on a flat surface.



4. When required, the lumbar support pad should be appropriately positioned.



2. Raise the foot section and align the plastic guide pins of the couplers. Lower the foot section into place. Depress pins, and slide both couplers forward into the locked position. Caution: Ensure that both locking pins have fully engaged the sliding coupler.

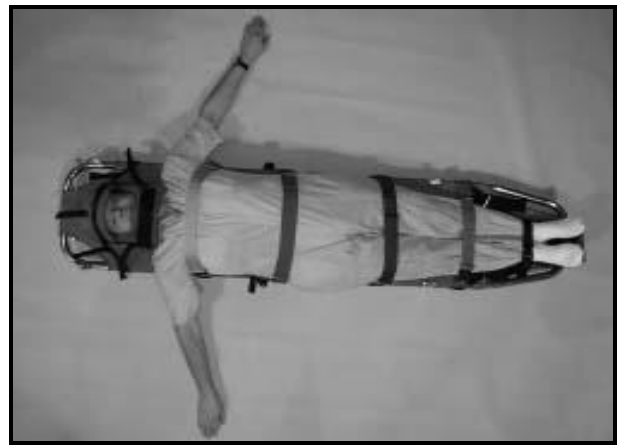


5. Secure the body restraint straps. Ensure that the gray strap crosses over the chest and under the arms.



3. Position patient on litter so that neck support pad is properly positioned under the neck. If required, secure the head restraint system into place. **Warning:** Do not over tighten the restraint straps, as this will cause discomfort or further injury to the patient.



6. Properly adjust the foot support strap.

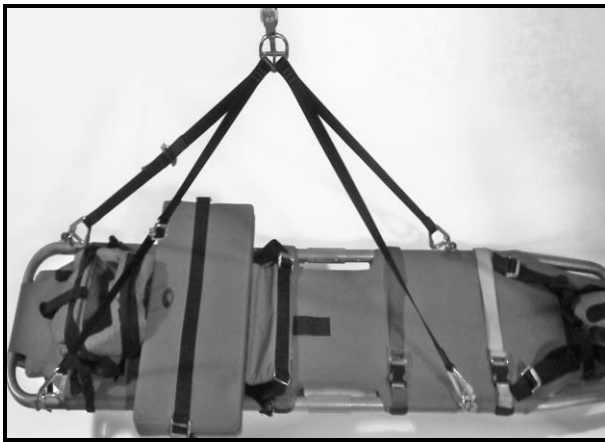

- For horizontal hoisting, attach the hoisting sling (LSC #192-S) to the litter as illustrated. The adjustable leg ends of the hoisting sling must be attached to the O-Rings at the head of the litter. Attach the fixed legs of the sling to the attachment O-Rings on the foot section of the litter.



- After use, accordion fold and re-stow the patient restraint straps. The upper two straps must be folded shorter than the lower straps. The end tab on the straps should extend slightly from the fold, but ensure that the main folds do not exceed beyond the edge of the litter frame or you will have difficulty closing the carrying case.



- Place the patient's arms down along the body. Place flotation pad on the chest and over the arms. Adjust and tighten the straps. Two universally sized carrying harnesses are supplied. The carrying harnesses allow the transporting personnel free use of their hands.



- For stowage, position the chest pad lengthwise on the upper section and push it up against the head restraint device. Unlock the couplers and fold the foot section over the head section. Place the folded litter into the open case, as per the instructions printed on the inside of the case.

MAINTENANCE & INSPECTION INSTRUCTIONS

- Every 30 days, and after each use, carefully inspect all fabric, webbing and hardware for wear, deterioration, cuts, abrasion and broken or missing parts. Repair or replace as necessary. Individual components are available from the manufacturer.
- After immersion in salt water, rinse all components with fresh water. Mild soap may be used to clean the fabric.
- If practical, avoid prolonged exposure to direct sunlight.

WARRANTY

LSC products are warranted to the first consumer purchaser to be free from defects in material or workmanship for a period of twelve (12) months. Please contact LSC for our complete Warranty information and Policies, or visit our website.

REPLACEMENT PARTS

#402-1S	CASE, (SINGAPORE Variant #402-2S)	#402-2	FRAME COVER
#402-3	FLOTATION PAD, FRONT	#402-4	FLOTATION PAD, BACK
#402-5	FOOT RESTRAINT STRAP	#402-6	LUMBAR SUPPORT PAD
#402-9S	INSTRUCTION CARD	#402-8	CARRING HARNESS
#402-12	STRAPS, FRONT FLOTATION PAD	#402-11S	FRAME ASSEMBLY, w/COUPLERS
#402-20	COUPLER LOCKING PIN	#402-13	NECK PAD
#402-22	INSERT GUIDE, FEMALE	#402-21	COUPLER, STAINLESS STEEL
#402-24	SPRING, STAINLESS STEEL	#402-23	INSERT GUIDE, MALE
#402-26	SCREW #6, 1/2" STAINLESS	#402-25	SCREW, 12/24, 3/4" STAINLESS

LIFESAVING SYSTEMS CORP.

220 ELSBERRY RD.
 APOLLO BEACH, FL 33572
 PHONE: 813/645-2748
 FAX: 813/645-2768
www.lifesavingsystems.com