



# OWNER'S MANUAL

## QUICK STROP

### #214

#### WARNINGS & CAUTIONS

1. Only appropriately trained rescue personnel shall use the Quick Strop.
2. The Quick Strop shall be used to hoist **uninjured** persons only.
3. The crotch (safety) strap shall be connected and adjusted during each lift.
4. Inspect all materials, webbing, and hardware for proper condition prior to use.
5. Ensure that the crotch strap is properly stowed prior to use.

#### DESCRIPTION

Originally designed for the US Coast Guard, the LSC Quick Strop is now available to all rescue activities. Constructed of MIL-SPEC webbing and stainless steel hardware, the Quick Strop provides a quick, safe means of hoisting **uninjured** personnel. The Strop can be placed under the arms, around the back or over the head of the survivor in one quick motion. To prevent the survivor from slipping out, a slide buckle slides down the Strop and attaches to an adjustable crotch safety strap, which is stored in a zipper pocket on the rear of the Strop. Instructions and pictorials are printed on the Strop. **Approved for use by the US Coast Guard.**

#### USING YOUR QUICK STROP

1. The Strop can be placed over-the-head and positioned around the back and under the arms. Or, remove the red-banded strap of the slide buckle to wrap around patient if patient cannot release hands for over-the-head donning.
2. If v-ring is removed, reinsert through slide buckle, pull slide buckle down both webbing legs to snug Strop around upper chest (under arms), with adjuster positioned in front of chest.
3. Remove crotch strap from pocket, pass between the legs and clip hook onto slide buckle.
4. Adjust as required, leaving slightly loose as strap will pull up during hoisting.
5. Keep arms down and folded across chest. See pictorial on inside of Strop.
6. Do not fold the padded area of the Strop.

#### INSPECTION

1. Inspect all materials, webbing, and hardware for cuts, abrasions, cracks, deterioration, and excess wear.
2. Remove from service any Strop with damage.
3. Every 6 months, apply a 600 lb. load to the extended Strop (v-ring to v-ring).

#### MAINTENANCE

1. Rinse thoroughly with fresh water, and air-dry the Strop after salt-water exposure.
2. Check the crotch strap adjustor and snap hooks for proper operation. Accordion fold and re-stow; ensure that the hook & loop closure matches properly on the crotch strap storage pocket and the crotch strap.
3. Mild soap and water can be used to hand wash the Strop.

#### WARRANTY

LSC products are warranted to the first consumer purchaser to be free from defects in material or workmanship for a period of twelve (12) months. Please contact LSC for our complete Warranty information and Policies, or visit our website.

#### LIFESAVING SYSTEMS CORP.

220 ELSBERRY RD.  
APOLLO BEACH, FL 33572  
PHONE: 813/645-2748  
FAX: 813/645-2768  
[www.lifesavingsystems.com](http://www.lifesavingsystems.com)