



OWNER'S MANUAL

#404 Medevac II Litter

WARNING: THIS LITTER SHOULD ONLY BE USED BY PERSONS WITH SUFFICIENT OPERATIONAL AND MEDICAL PROPER PATIENT HANDLING.

STOWING:

When litter is not in use the proper stowage procedures should be taken:

1. Buckles strapped across the litter with minimal slack, see Figure 1.
2. Flotation secured to the frame, see Figure 1.
3. If practical store to avoid prolong exposure to direct sunlight.
4. **Recommended:** Use #409 Litter Cover while stowed to protect from unnecessary and unintentional exposure and damages, see Figure 2. Zipper should be fully closed during stowage.

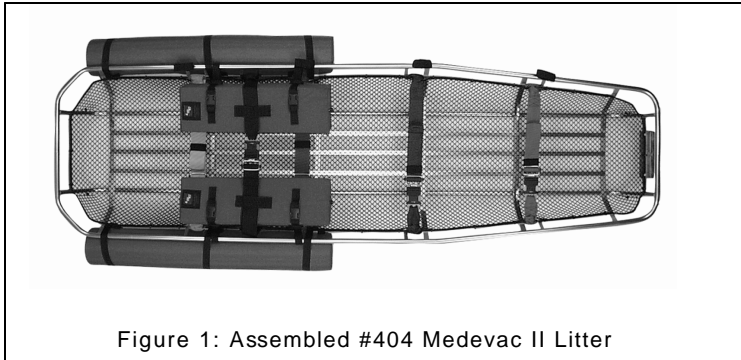


Figure 1: Assembled #404 Medevac II Litter



Figure 2: Medevac II Litter Stowed in Case

INSPECTION

Every 30 days, and after each use, carefully inspect the following:

1. Frame assembly for broken welds, bends (deformation), corrosion and abrasion.
2. Check float covers, floats, the plastic net liner, hardware, and restraint straps for cuts, tears, abrasion and any other visible damages. Repair or replace as required.

Remove Litter from service if inspection identifies any damage or abnormal conditions indicated above. Repair or replace accordingly.

REPAIR & MAINTENANCE

Stainless Steel: All weld repairs shall be TIG welded by qualified welders only. **Only** minor repairs should be performed in the field.

Titanium: **Not recommended** to make field repairs, titanium welding requires specialized equipment and training.

1. After use or as required, thoroughly wash the entire Litter with fresh, soapy water. Let litter thoroughly dry.
2. Remove any flotation or medevac boards prior to washing frame. Wash items separately with fresh, soapy water. Let floats, straps, and boards thoroughly dry.
3. Litter mesh liner replacement: Remove liner by cutting the plastic cable ties. Align liner and secure with new cable ties. Install cable ties such that cut ends will be facing outboard of the Litter. Cut off excess length of all ties squarely.
4. **IMPORTANT:** Damage to tubular frame members and welds, shall be reviewed by the manufacturer for repair and serviceability determination.

REPLACEMENT PARTS

#439	Mesh Liner	#140	Patient Restraint Straps, set of 5
		#150	Retainer Straps

WARRANTY

LSC products are warranted to the first consumer purchaser to be free from defects in material or workmanship for a period of twelve (12) months. Please contact LSC for our complete Warranty information and Policies, or visit our website.

LIFESAVING SYSTEMS CORP.
 220 ELSBERRY RD.
 APOLLO BEACH, FL 33572
 PHONE: 813-645-2748 FAX: 813-645-2768
 www.lifesavingsystems.com