



OWNER'S MANUAL

MEDEVAC BOARD

#450

WARNINGS & CAUTIONS

All personnel must be properly trained and qualified by the appropriate medical organization prior to using the Medevac Board.

The board & components must be thoroughly disinfected and cleaned or replaced after contact with body fluids.

The frame screw coupler must be completely tightened prior to use.

Inspect all straps, hardware, and accessories for proper condition prior to use.

DESCRIPTION

The Medevac Board is designed to provide effective, quick patient immobilization. It is a two-piece, full length, aluminum backboard. It is designed for compact stowage and shaped to fit inside the LSC #404/406 litters. It is ideally suited for confined space rescue. The board halves are constructed with stainless steel screw couplers for strength and rigidity. Constructed of heavy gauge aluminum, the Medevac Board offers a load capacity in excess of 300 lbs. The nonporous surface makes it easy to clean and impervious to blood and other body fluids. Strategically placed oversized, replaceable hand straps make it easy to carry and maneuver. Comes complete with replaceable color-coded patient restraint straps, which feature quick-close buckles, and a disposable head restraint. The Medevac Board is easily separated into two halves for space-saving storage and comes with a protective storage/carrying case.

MEASURES: 16.25" x 76.25" x 1.5" assembled

WEIGHT: 19 lbs

USING YOUR MEDEVAC BOARD

Only personnel trained in the organization's current medical protocol should be allowed to use the Medevac Board. Prior to each use, the Medevac Board should be inspected for the following:

1. Inspect panels, frame, couplers, and restraint straps for excessive wear, cleanliness, corrosion, cuts, security, and proper operation.
2. Inspect Head Restraint device for cleanliness.

REPLACEMENT PARTS

#450-1	STRAP & HANDLE SET	#450-2	DISPOSABLE HEAD RESTRAINT
#450-3	MEDEVAC BOARD CASE	#450-4	FOOT STRAP

WARRANTY

LSC products are warranted to the first consumer purchaser to be free from defects in material or workmanship for a period of twelve (12) months. Please contact LSC for our complete Warranty information and Policies, or visit our website.

LIFESAVING SYSTEMS CORP.

220 ELSBERRY RD.

APOLLO BEACH, FL 33572

PHONE: 813/645-2748

FAX: 813/645-2768

www.lifesavingsystems.com